

## Saliva Alcohol Test for Zero Tolerance Testing



**Alco-Screen® 02 is the most simple, cost effective method of monitoring for alcohol consumption in your Zero Tolerance testing program.**

Alco-Screen® 02 is so effective that it has been tested and approved by the US Department of Transportation (DOT) for required testing of all transportation and safety sensitive employees for blood alcohol concentrations above the federally mandated zero tolerance level of 0.02%.

Alco-Screen® 02 is a simple one-step saliva screening test that works in a clean, non-invasive manner and provides results within 4 minutes. Simply wet the test pad with saliva; development of a line on the test pad within 4 minutes indicates a blood alcohol concentration exceeding 0.02% as shown below:



**Positive Result:** Blood alcohol above 0.02%



**Negative Result:** Blood alcohol less than 0.02%.

This simple, highly effective test allows you to tell if the individual being tested is under the influence of alcohol, putting you in control.

Inappropriate consumption of alcohol can be a problem in many family, occupational, institutional and safety sensitive situations.

Alco-Screen® 02 provides you with an easy yet effective tool in identifying alcohol consumption. And it is the most cost effective alcohol testing method available.